



BEL-RED SLEEP DIAGNOSTIC CENTER
1414 116th Ave NE Suite F, Bellevue, WA 98004
Phone: 425-451-8417 | Fax: 425-455-4089

CPAP/BIPAP CLEANING INSTRUCTIONS

MASK CUSION/PILLOW:

- Wipe the mask cushion or pillow and the forehead pad daily with a damp washcloth or paper towel. DO NOT use alcohol-based solutions or those containing hand creams.

MASK & HEADGEAR:

- Wash the mask and headgear at least once a week with a mild soap. Rinse well with warm water and airdry (leave mask and headgear assembled to prevent frustration in putting it back together and getting a good fit after removing them). Avoid placing mask and headgear in direct sunlight.

FILTERS – RESPIRONICS

- The White Ultra Fine is disposable. The smooth side faces outward. When the rough side starts to show “gray,” it is time to change the filter.
- The pollen filter (dark gray foam) is washable. Wash the filters once a week with mild soap. Rinse well with warm water and air dry. If the filter is still damp when putting it back in the machine, use the extra filter provided.

FILTERS – RESMED

- The filter(s) in this manufacturing machine is disposable. Replace filters every 1-3 months.

TUBING

- Handwash the tubing with mild soap and rinse well with warm water weekly. Airdry before reattaching to the unit. Avoid placing tube in direct sunlight.

HUMIDIFIER CHAMBER

- Empty any remaining water from the chamber and wash with mild soap once a week. Airdry and avoid placing chamber in direct sunlight.
- Disconnect the tubing from the unit when you turn the power off – this will help eliminate the accumulation of condensation.